



Finger Bandage, Plastic Finger Stall & Applicator

DIRECTIONS

Make sure the wound is clean and dry before the dressing is applied.

Once the dressing has been applied fully extend the elastic strap on the finger stall.

Carefully slide the stall over the bandage.

Adjust the strap for a comfortable fit and secure.

Not intended to be used for prolong periods or continual use.

Where only when dressing is likely to become soiled or damaged and remove when sleeping.

Types of Bandages

Closed End Bandage

Protects injured fingertips and damaged nails.


Open End Bandage

Protects injuries sustained to the length of the finger.

Tied Bandage

Protects injuries to the length and tip of the finger and provides additional support

Closed End Bandage


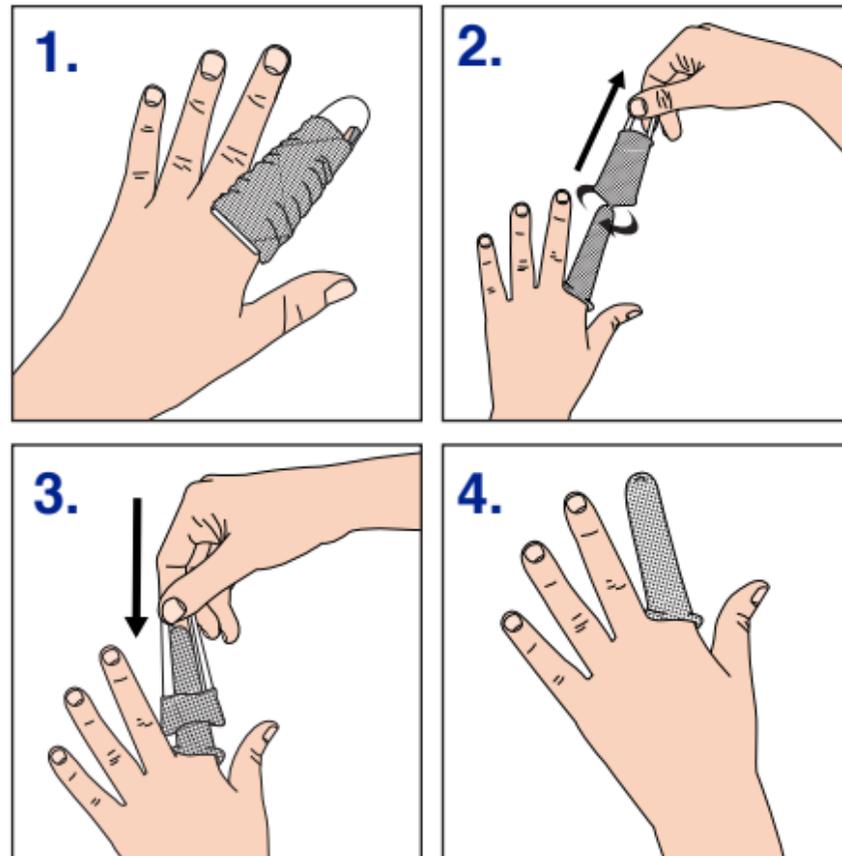


Closed End Bandage

INSTRUCTIONS

1. Measure and cut the bandage to approx. 3 times the length of the finger to be treated. Loosely gather the bandage onto the applicator and place the applicator over the finger to be treated so the open sides are between the fingers.
2. Hold the bandage at the base of the finger and slowly withdraw the applicator. Once the base of the applicator is just above the tip of the finger twist the applicator round once.
3. Push the applicator back down the finger so the bandage is double in thickness. Remove the applicator.
4. Secure the bandage in place using adhesive tape.

Open End Bandage


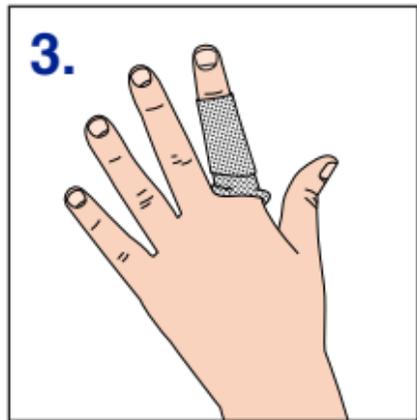
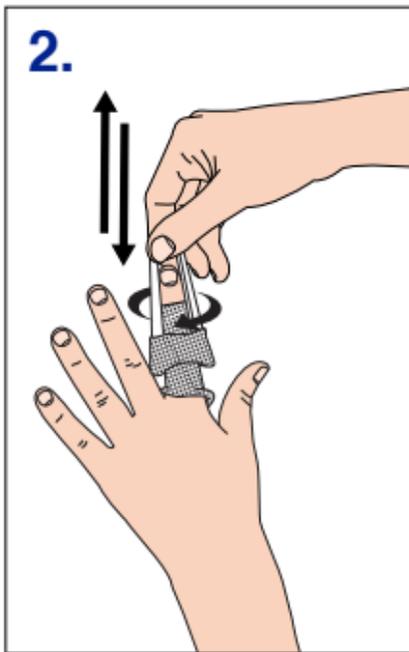


Open End Bandage

INSTRUCTIONS

1. Measure and cut the bandage to approx. the length of the finger to be treated. Follow the instructions for the closed end bandage. However when withdrawing the applicator stop before you get to the end of your finger.
2. Twist the applicator half a turn. As with the closed end bandage push the applicator back towards the base of the finger.
3. Secure the bandage in place using adhesive tape.

Tied Bandage

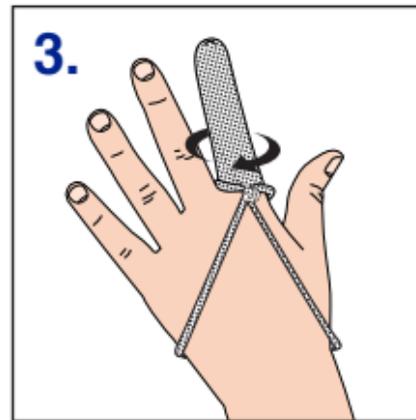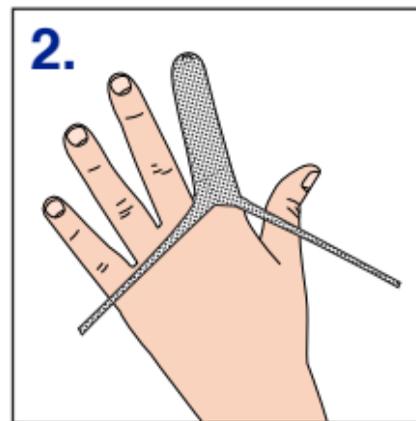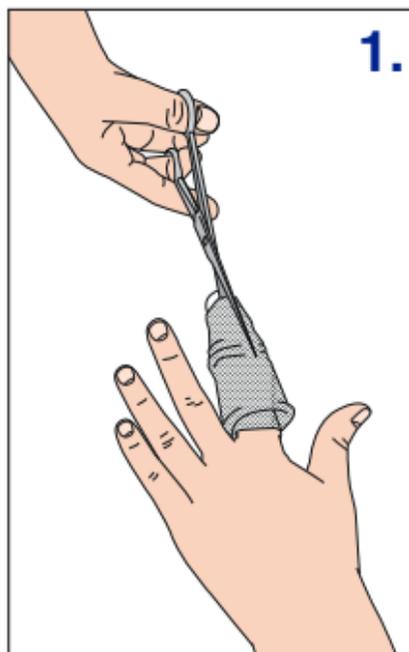

Tied Bandage

INSTRUCTIONS

1. Measure and cut the bandage 4 to 5 times the length of the finger to be treated. Follow the instructions for the closed end bandage. However, once the second layer of the bandage has been applied, keep the remainder on the applicator and cut down each side.
2. Remove the applicator and tie the two loose strips together at the base of the bandage to prevent it from tearing.
3. Wrap the two strips around the wrist and tie securely.